

The Most Popular & Profitable Aesthetic Procedures Today

TRIO XT

FOCUSED TRI-POLAR ALTERNATING DEPTH RF

PATIENT
EXPECTATION
PRESENTATION

AESTHETIC
COSMETIC
PROCEDURE



SIGNS OF AGING • SKIN TIGHTENING
WRINKLE AND FINE LINE REDUCTION
BODY CONTOURING & SHAPING
CELLULITE REDUCTION
COLLAGEN REMODELING

BEGIN A NEW LIFE WITH A MORE YOUTHFUL APPEARANCE

SYBARITIC®
Global Aesthetic Solutions

TREATMENTS

TRIO XT

FOCUSED TRI-POLAR ALTERNATING DEPTH RF

THE TRIO XT TREATMENTS

Almost everyone wishes for a more sculpted, smoother looking body or a reduction in wrinkles and fine lines on their face. With the versatile TRIO XT radio frequency system, you can choose from facial or body skin tightening treatments to suit your individual aesthetic goals and objectives.

The TRIO XT offers numerous benefits over other aesthetic procedures. Unlike lasers or Intense Pulsed Light (IPL), TRIO XT treatments can be delivered on anyone with any skin type (Fitzpatrick Skin Type Scale I-VI) or tanned patients. The procedure requires no post procedure "downtime" for recovery. You can resume your regular daily responsibilities immediately following each session. TRIO XT sessions are accumulative, so the results will increase with each procedure. Packages typically range between 4-8 comfortable sessions and can be combined with other aesthetic treatments for maximum results.

HOW DOES THE TRIO XT WORK?

The system delivers a very safe photon of energy called radio frequency (RF). When RF is introduced onto the surface of the skin, the electrons from the RF energy penetrate down to the deep dermal layers where collagen is present. Heat is generated by the skin's own resistance to the polarity of the electrons. As the resistance continues, the tissue is carefully heated to the point where a reaction occurs causing the collagen's triple-helix molecular structure to unravel and shrink, resulting in the initial collagen denaturation process. Long term, the results are more dramatic. The body's natural wound-healing process begins after each treatment, stimulating the formation of new collagen growth for better looking skin.

THE TWO (2) TYPES OF RESULTS

1. IMMEDIATE: Initial skin tightening as a result of collagen denaturation.

A. For the face, this means wrinkles and fine lines are diminished with each session, while your skin will appear smoother and tighter.

B. For the body, you skin will look and feel tighter, while the circumference of the target area being treated should present a measureable reduction in size.

2. LONG TERM: Following the immediate collagen denaturation process that initially shrinks the collagen in the dermis of your skin, the body's natural wound-healing process stimulates the formation of new collagen growth 10-12 weeks after each treatment. The long-term result is the most dramatic, providing about 66% of the final result.



THE PROCEDURE

TARGET AREA	LENGTH OF PROCEDURE	NUMBER OF PROCEDURES
Full Face	40 minutes	4-6
Partial Face	20 minutes	4-6
Periorbital Area	10 minutes	4-6
Forehead	10 minutes	4-6
Neck	20 minutes	4-6
Abdomen	40 minutes	6-8
Thighs /Legs	40-60 minutes	6-8
Cellulite Reduction	40-60 minutes	6-8
Arms	30 minutes	4-8
Back	20-30 minutes	4-8

WHAT CAN I EXPECT DURING EACH PROCEDURE?

First you will complete an informed consent and review the procedure contraindications with the physician. At the procedure time, you will be asked to remove all jewelry and metal from the area to be treated. Depending on which part of your body is being treated, you may need to remove some articles of clothing to allow for better treatment access. Once you are comfortably on the treatment table, the technician will gently cleanse the treatment area. To provide better comfort and the best energy conduction, the technician will apply a generous coat of liquid gel on your skin. As the procedure begins, you will feel some warmth on your skin, this is completely normal. The technician will ask you about your comfort level until they have adjusted the energy and contact cooling (if applicable) settings to the proper level. The procedure should be moderately comfortable. During the course of each procedure, there is a fine line between comfortable heat and the clinical end point. For this reason, you may occasionally feel a sharp peak of heat similar to touching something very hot. There is no need to worry, this hot sensation will not last longer than a split second. There are no long-term effects to these peaks and they will not negatively affect the outcome of your session. They are part of the normal procedure.

WHAT CAN I EXPECT IMMEDIATELY AFTER EACH SESSION?

The treated facial area may appear dark pink to red in areas that are either sensitive or received the most time during the procedure. The face will generally feel warm for about 30-60 minutes. Most patients simply clean off the gel, reapply their makeup and leave immediately following the procedure. If the area feels uncomfortably warm, a cool towel or a liquid gel masque such as the **INFUZION** gold liquid masque can be applied for about twenty minutes to help calm down the warmth before the patient leaves.

The treated body areas are generally less sensitive than the face. Most patients experience a minimal amount of sensitivity and simply clean off the gel, get dressed and leave immediately following the procedure.



Photos courtesy of Dr. Alejandra Builes M.D.

IMMEDIATELY AFTER TREATMENT

Immediately following a facial procedure you will notice a slight pink area around the eyes.

AFTER CARE & RESULTS

HOW DO I CARE FOR THE TREATED AREA BETWEEN PROCEDURES?

The body is very resilient. As part of the end result, healing will begin immediately following each session. Even though you may not feel or see it, the skin is re-healing itself from the thermal "damage" and the collagen remodeling process is starting. During the course of the treatment program, stay well hydrated by drinking plenty of water every day. Radio frequency achieves better resistance for more thermal efficiency with hydrated skin. You will see an improved appearance of your skin and the natural accumulative results will be more dramatic when you stay well hydrated and apply a daily skin moisturizer. One of the great benefits of **TRIO XT** treatments is that you can wear makeup and resume your normal activities right after every session.



Photos courtesy of Dr. Alejandra Builes, M.D.



BEFORE

Photos courtesy of Dr. Alejandra Builes M.D.



AFTER six (6) treatments

Photos courtesy of Dr. Alejandra Builes M.D.

“ I have just experienced your TRIO RF system in Singapore and am amazed at the results after only 2 sessions!
Yours sincerely, ”

— Kate Bean
Melbourne Australia

WHEN WILL I SEE THE RESULTS?

Most people will see a visible or measurable result after even one procedure.

As the treatment guide on the previous pages illustrates, multiple treatments are necessary for the maximum result.

The most dramatic result will appear gradually 10-12 weeks following your last procedure.



Sybaritic, Inc., Global Aesthetic Solutions

9220 James Avenue South Minneapolis, Minnesota 55431 · www.Sybaritic.com

Toll Free (800) 445.8418 Tel (952) 888.8282 Fax (952) 888.8887 · info@sybaritic.com

© 2008 Sybaritic, Inc., Mpls, MN TXT.6.1.811.1.I